


HeartSine samaritan® PAD Upgrader (1.4.2/3.2.0)


User Instructions

1

Following download and installation of PadUpgrader:

Click on the PADUpgrader Icon  located on computer desktop or else press the Start menu button and select PADUpgrader

After a few seconds, the 'PAD Upgrader-(1.4.2/3.2.0)' window should appear.

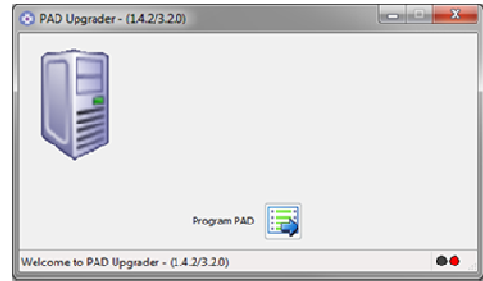


2

Attach PAD to computer's USB port.

After a few seconds, the samaritan® PAD icon will appear and the connection indicator will turn green.

samaritan® PAD icon
connection indicator

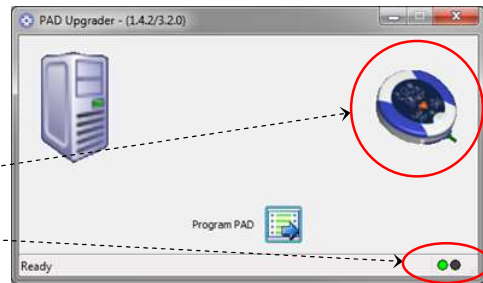


3

Perform the Upgrade.

Click the 'Program PAD'  button to begin the upgrade.

Follow any on-screen instructions.

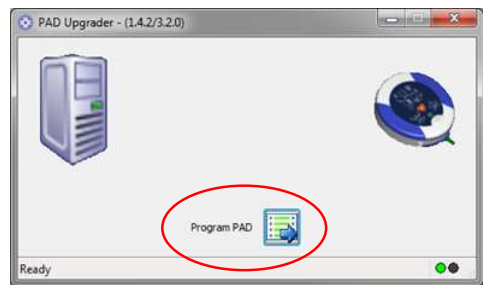


4

Return Proof of Upgrade.

After a successful upgrade, a pdf file will be created on your Desktop. Please return this in one of the following ways:

1. E-mail to support@heartsine.com
2. Print out and post to: HeartSine Technologies
Canberra House
203 Airport Road West
Belfast
BT3 9ED
United Kingdom
3. Fax to +44 (0)28 9093 9401

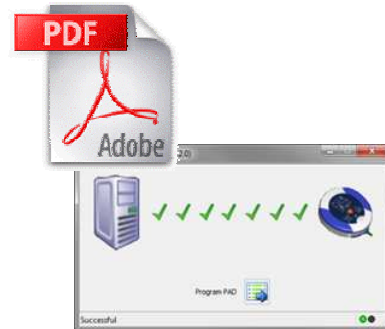

Important Warning



If an error occurs at any point during the upgrade, the samaritan® PAD **must not be used** until the upgrade has been successfully completed.

If you have any queries, please contact your distributor or e-mail support@heartsine.com