



Universal Trainer Configuration Tool

Operating Instructions Version 1.0.1 – Samaritan PAD Trainer 300P / 500P

Trainer Configuration Tool

The *Trainer Configuration Tool* is provided for the purpose of changing scenario, volume, pairing, guidelines, or language settings on the PAD Trainer device via a USB connection.

Please Note: Driver files are required to be installed on any PC wishing to make use of the USB facility. These will be installed along with the *Trainer Configuration Tool*

Installation

The *Trainer Configuration Tool* is installed by running the TrainerConfig_setup.exe installation program. The user will be prompted for an installation directory to which all the necessary files will be installed.

Password Protection


The *Trainer Configuration Tool* will prompt the user for a password whenever any device settings are changed. By default this password is 'password'. It can be changed by clicking the settings button in the upper right of the initial screen.

Changing Settings


To change the settings on the PAD Trainer click on the 'Samaritan PAD Trainer' button on the main screen:



On the following screen, click on 'Settings'. There will be a brief pause as the PAD Trainer initialises. After this, the following screen will be displayed:

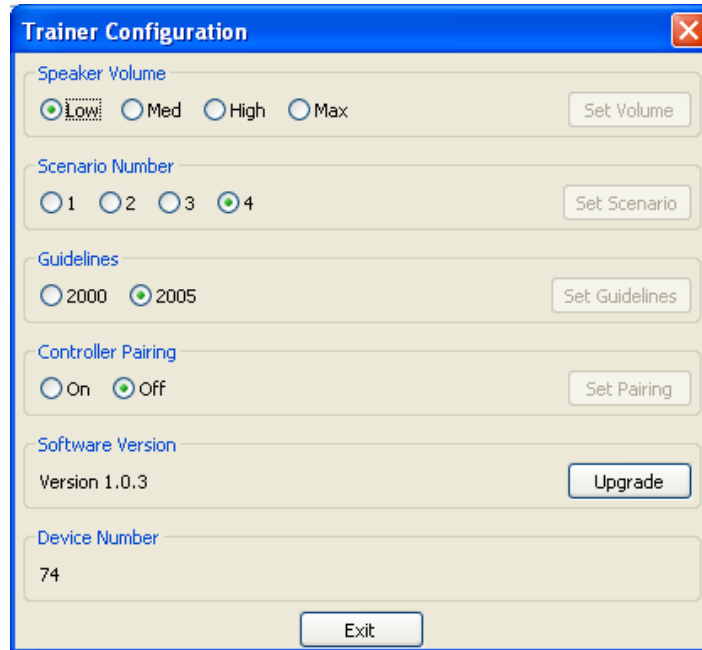

To change any settings the appropriate option should be selected. Once selected, the relevant 'Set' button will be activated on the right hand side of the screen. Clicking this button will both change the setting and de-activate the 'Set' button until another option is chosen. On clicking any 'Set' button the user will be prompted for a password.

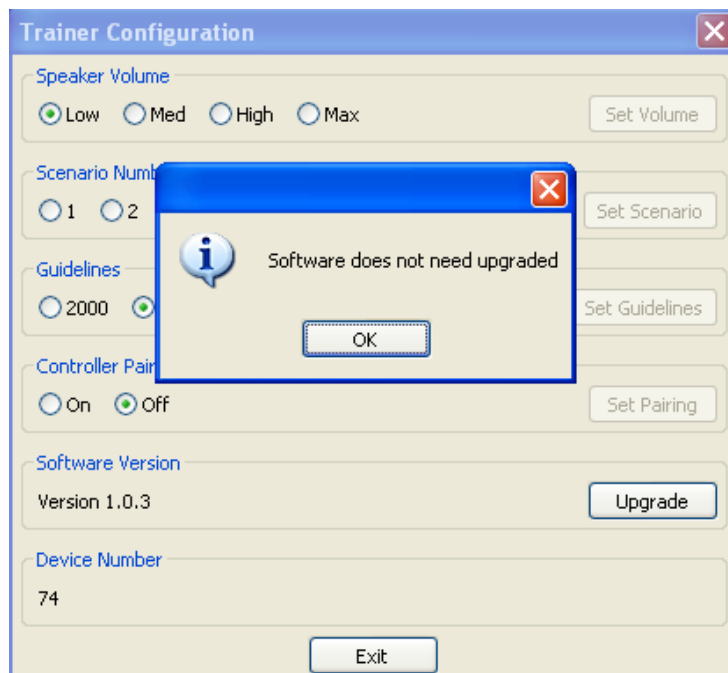

The current revision of device software will be contained within the website download. To upgrade a device click the 'Upgrade' button. If the device already contains the current revision of software the message 'Software' does not need upgraded" will be displayed.



Remote Control – Pairing with Device

Remote control handsets are paired with a device by way of a pairing ID enabling two trainer devices to be used side by side. The remote from one device will not affect the second device and vice versa.

Since there are a limited number of ID's a user may find himself in possession of 2 devices which share the same ID and whose remote controls will then operate both devices. Where this problem arises the user may perform the pairing procedure detailed below.

This procedure will pair together a remote and device using a randomly selected ID.

Pairing Procedure

- Plug the supplied USB cable into the device
- Using the Universal Trainer Configuration tool enable "Controller Pairing"
- Disconnect the USB cable and turn ON the device with the GREEN ON/OFF button. The chest LEDs will illuminate indicating pairing mode.
- Remove the handheld remote battery cover
- Hold down the remote RED OFF button and lift one of the batteries out.
- Replace the battery and the remote will pair with the device – 12 audible beeps will be heard if successful
- Release the OFF button and replace the battery cover once the beeps begin
- When the beeps have finished press the RED OFF button again
- The Trainer Device will power OFF
- The pairing procedure is now complete

Please Note: Inserting the batteries into the handheld remote whilst pressing the RED OFF button will cause the remote to take a new transmission ID. If this occurs accidentally when the device is not in pairing mode it will be necessary to complete the above procedure in full to pair the device and remote.

HeartSine Technical Support:
Americas: 1-215-860-8100, ext 107
Email: info@heartsine.com
Europe/International: 00442890 939 404
Email: support@heartsine.com